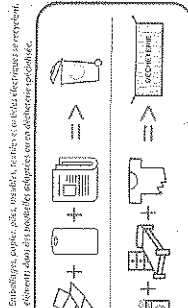
Imported by - Importé par:
GD Import (Genius Ideas & Wellis)
190 chemin de la Frayère - 06530 Peymeinade - France
www.gdimport.com
www.genius-ideas.com
www.wellis-care.com

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GB - EAR CLEANER - 013920 / 013921

Instructions:

Image 1: Open the battery cover at the bottom of the device and install 2 AA batteries into position (as shown) then close the cover.

Image 2: Put the silicon ear plug on the tube of the unit (as shown) and ensure that it is firmly placed on the tube.

Image 3: Switch the unit on, place the tube in the ear and slightly turn in order to eliminate earwax and other impurities.

Attention: The silicone head must be used in order to ensure comfortable and secure handling. You should only use the unit on healthy ears (not sick, painful or inflamed). The silicon heads and the filter can be cleaned with soap and water.

Tip if suction problem: Following rare manufacturing errors, sometimes the device is drawing instead of breathing. In such a case, simply switch the polarity of the batteries (2 + becomes - and - becomes +).

Danger Associated with Batteries

- Always replace all batteries. Do not use new and used batteries at the same time. Do not use any different battery types, brands or batteries with different capacities. When changing the batteries, check the polarity (+/-).
- Take the batteries out of the unit if they are empty or if you do not use the unit for a prolonged period of time. Thus you will prevent damages that could occur if the batteries leak.
- Non-rechargeable batteries must not be recharged.
- The terminals must not be short-circuited.

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GB - This product is labelled in accordance with European Directive 2012/19/UE concerning used electrical and electronic appliances (WEEE). Do not dispose of electrical appliances as unsorted municipal waste, use separate collection facilities. Contact your local government for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

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