



**MOPEDIA®**

ENGLISH

## **WALKING STICKS**

---

## **INSTRUCTION MANUAL**

## INDEX

<b>1. CODES.....</b>	<b>PAG.3</b>
1.1 Folding walking cane codes .....	pag. 3
1.2 Aluminum walking canes height adjustable codes.....	pag. 3
<b>2. INTRODUCTION.....</b>	<b>PAG.3</b>
<b>3. INTENDED USE.....</b>	<b>PAG.3</b>
<b>4. DECLARATION OF CONFORMITY .....</b>	<b>PAG.3</b>
<b>5. GENERAL WARNINGS .....</b>	<b>PAG.4</b>
<b>6. SYMBOLS .....</b>	<b>PAG.4</b>
<b>7. GENERAL DESCRIPTION .....</b>	<b>PAG.4</b>
7.1 Height adjustment .....	pag. 4
<b>8. BEFORE USE .....</b>	<b>PAG.4</b>
<b>9. WARNINGS FOR USE .....</b>	<b>PAG.5</b>
<b>10. HOW TO USE.....</b>	<b>PAG.5</b>
<b>11. MAINTENANCE .....</b>	<b>PAG.5</b>
<b>12. CLEANING AND DISINFECTION .....</b>	<b>PAG.5</b>
12.1 Cleaning.....	pag. 5
12.2 Disinfection .....	pag. 5
<b>13. CONDITIONS OF DISPOSAL .....</b>	<b>PAG.6</b>
13.1 General conditions of disposal .....	pag. 6
<b>14. TECHNICAL FEATURES .....</b>	<b>PAG.6</b>
14.1 Aluminum folding walking cane.....	pag. 6
14.2 Aluminum height adjustable walking cane .....	pag. 6
14.3 Spare parts .....	pag. 6
<b>15. WARRANTY .....</b>	<b>PAG.7</b>
15.1 Repairs .....	pag. 7
15.2 Spare parts.....	pag. 7
15.3 Exempt clauses.....	pag. 7



REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

## 1. CODES

### 1.1 Folding walking cane codes

**RP145NC** Aluminum folding walking cane with T-handle - Black color

**RP145NL** Aluminum folding walking cane with T-handle - Black color

### 1.2 Aluminum walking canes height adjustable codes

**RP146** Aluminum height adjustable walking cane T-handle - Black color

**RP148** Aluminum height adjustable walking cane T-handle - Black color

**RP153D** Aluminum height adjustable walking cane anatomical right handle

**RP153S** Aluminum height adjustable walking cane anatomical left handle

## 2. INTRODUCTION

Thank you for purchasing a **MOPEDIA** walking canes by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

## 3. INTENDED USE

The MOPEDIA by Moretti walking sticks are aids manipulated by one arm adapted to support and / or help the mobility of people with mobility problems.

### CAUTION!



- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of WALKING STICKS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:





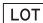



1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

- Read carefully the entire manual before using the device.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the walking canes.
- Make sure that the product is delivered complete and keep it packed away from heat sources.
- Service life- the device use limit is defined by the wear of the parts.
- Do not allow children to play or operate the device.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

## 7. GENERAL DESCRIPTION

MOPEDIA walking canes are made of lightweight anodized aluminum, wooden, plastic or sponge handle and non-slip rubber tip. Adjustable height every 2.5 cm.

These devices are useful to ensure a safety of support and confidence. The walking canes sticks also make it easy to stay upright and walk. Facilitate the discharge of the lower body segments, stabilize the person in the problems of balance and support when there are problems force.

### 7.1 Height adjustment

Press the button, slide the cane and bring it to match the holes in the height required and the maneuverability of the patient. Replace the stopper on the hole to lock the cane and make sure that the button is fully penetrated into the hole. Maximum patient's weight 100kg for folding canes, 110 kg for all the others.

## 8. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

## 9. WARNINGS FOR USE

- Use extra caution on wet and slippery surfaces.



### WARNING!

DO NOT use with patient who exceed the weight indicated in the product label and in the technical features chapter of the present manual.

- Avoid to leave the device in contact with water for long periods of time.
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out.
- In case of doubts about use, adjustment or maintenance, contact your dealer.
- Keep it away from children to avoid any possible risks.

## 10. HOW TO USE

**Note:** Refer to the safety instructions in this manual

The use of the walking canes varies from case to case. For this you need a minimum of training by qualified personnel with reference to information on the type of step or support type. The operator during patient care is always positioned on the side of the body more weak and sick.

## 11. MAINTENANCE

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

For periodic inspection we mean an examination of the following parts:

- The walking cane frame
- Check the wear of rubber tip.

In case of repair are to be used only approved parts and accessories

## 12. CLEANING AND DISINFECTION

### 12.1 Cleaning

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove more difficult dirt. Dry thoroughly before use. Never use water with temperature above 30°C. Never use jet water or steam washing machine.



### WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces

### 12.2 Disinfection

if you need to disinfect the device use a common disinfectant cleaner.

**Note:** Never use acids, alkalis or solvents such as acetone

### 13. CONDITIONS OF DISPOSAL

#### 13.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

### 14. TECHNICAL FEATURES

#### 14.1 Aluminum folding walking cane

Codes	Max weight	Height adjustable	Handle	Frame
RP145NC	100 kg	From 77,5 to 87,5 cm	T-wood	In anodized aluminum
RP145NL	100 kg	From 84,5 to 94,5 cm	T-wood	In anodized aluminum

#### 14.2 Aluminum height adjustable walking cane

Codes	Max weight	Height adjustable	Handle	Frame
RP146	110 kg	Da 79 a 102 cm	T-wood	In anodized aluminum
RP148	110 kg	Da 79 a 102 cm	T-wood	In anodized aluminum
RP153D	110 kg	Da 79 a 102 cm	Anatomical right	In anodized aluminum
RP153S	110 kg	Da 79 a 102 cm	Anatomical left	In anodized aluminum

#### 14.3 Spare parts

##### RV7029 (2pz)



Brown rubber tip  
Ø 18 mm

##### RV7028 (2pz)



Black rubber tip  
Ø 18 mm

## 15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of ( two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed.

The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti ), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 15.1 Repairs

**Warranty repairs** In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

**Repair not covered by warranty** A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

**Non-defective device** The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

### 15.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

### 15.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE****Product** \_\_\_\_\_**Purchased on (date)** \_\_\_\_\_**Retailer** \_\_\_\_\_**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_**Sold to** \_\_\_\_\_**Address** \_\_\_\_\_ **Town/city** \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

**www.morettispa.com** email: **info@morettispa.com****MADE IN P.R.C**

**NOTES**

## NOTES

# ENGLISH

**NOTES**

**MORETTI S.p.A.**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)