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moem

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EC DECLARATION OF CONFORMITY

Declara que el producto / *This is to certify that the product*

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Complies with the requirements of RD 1591/2009 of 16th October, that regulates medical devices and implementing Directive 93/42/ECC, modified by Directive 2007/47/EC.

Declaration in accordance with Annex VII of RD 1591/2009. Medical devices class I

Reference standard: UNE-EN 12183 “Manual wheelchairs. Requirement and test methods”.

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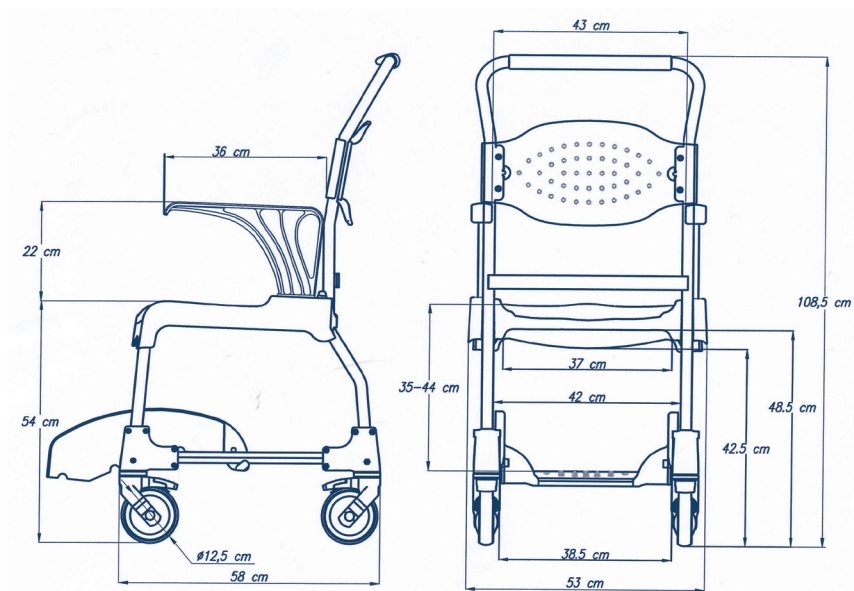
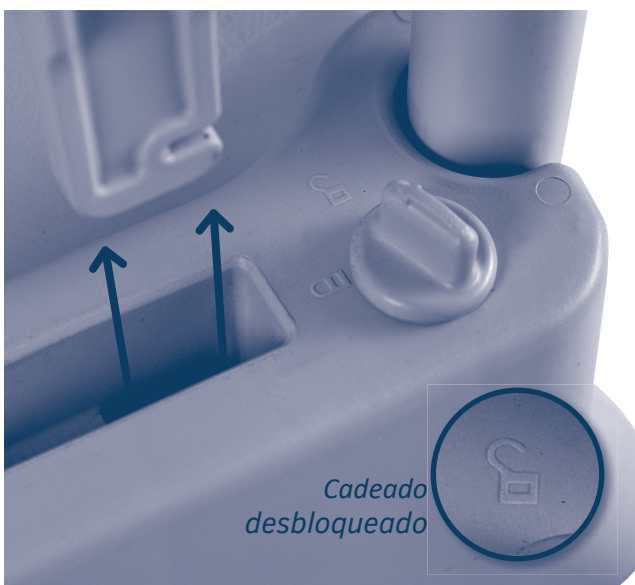
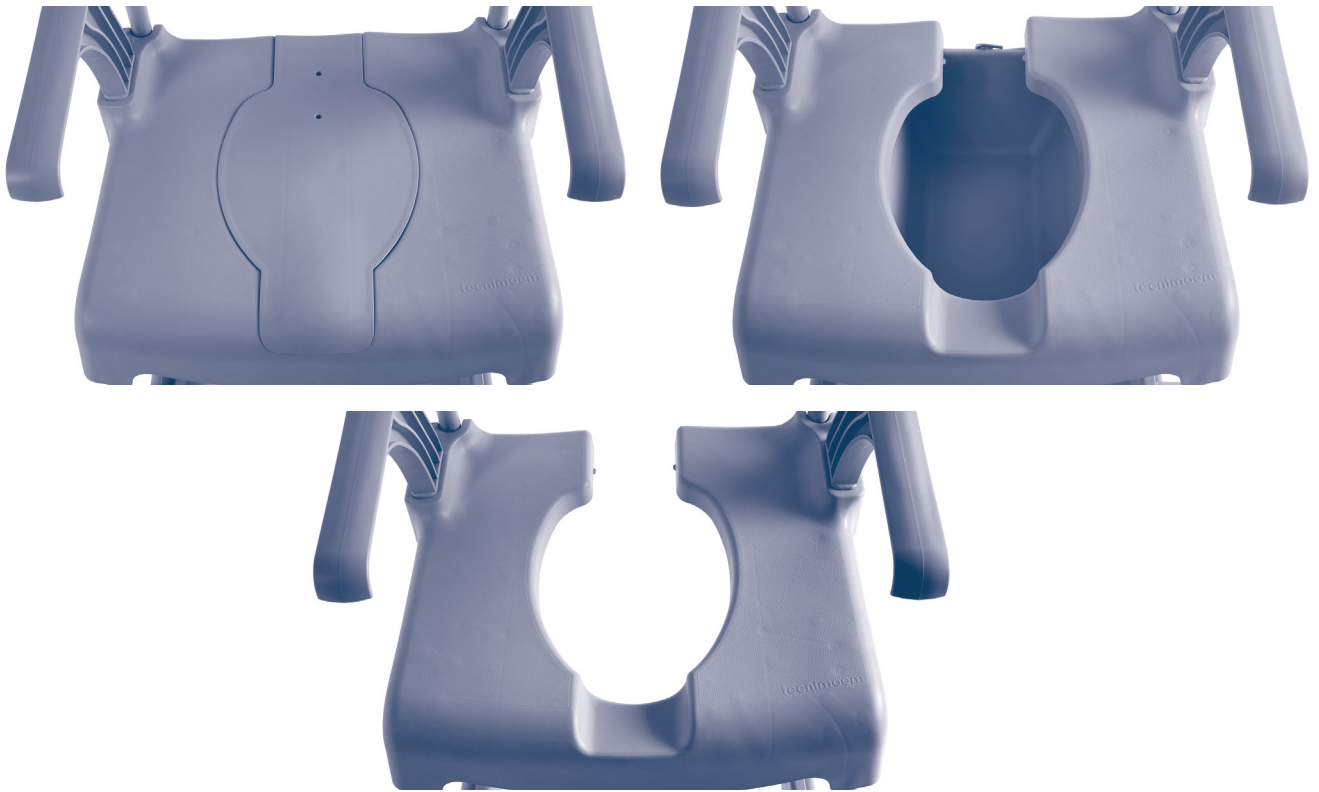
If the chair is occupied, use the care assistant's hand-hold with the chair facing backwards.

Brake the chair (image B 1)

Lock the wheels by stepping on the brake pedal. The chair is more stable if the wheels are turned outwards.

Transfer

Make sure the brake is on the wheels before starting to transfer the patient to another support (toilet, bed...). Remove the armrest beforehand if necessary.

4/ ARMREST See image C (page 4)

Armrest can be extracted using the wheel.

5/ SEAT COVER See image D (page 4)

Easy to extract.

6/ FOOTREST See image E (page 5)

Detachable and easy to use.
Adjustable to 5 positions.

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10/ TECHNICAL SERVICE

Tecnimoem 97, S.L. has a dealer network for the sale of all its products. In the event of any problem, the end user must contact the dealer from which the product was purchased. Tecnimoem trains its dealers and provides them with sufficient spare parts to maintain the products.

TECNIMOEM 97, S.L. has a policy of continually improving its products. For that reason, device specifications may be changed without notice.

11/ END OF LIFE CYCLE

After the article's life cycle it should be taken to a recycling point. In this way you can help to protect the environment.

www.tecniemoem.com

Sistema de
Gestión
ISO 9001:2015
ISO 14001:2015
www.tuv.com
ID: 910054418

Tecniemoem 97, S.L. Ctra. Logroño-Mendavia, Km. 5
31230 Viana (Navarra) - ESPAÑA (Spain)
Tel. +34 948 646 213 - Fax. +34 948 646 380
tecniemoem@tecniemoem.com