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INTENDED USE

The Rollator is a mobility aid intended to be used by people who are experiencing generalized weakness, difficulty with balance, a lower extremity injury or surgery that need additional support to maintain balance and stability while walking outdoors or used for rehabilitation.

SAFETY INSTRUCTIONS

USER: BEFORE using the Rollator, read this manual and save for future reference.

CAUTION: DO NOT attempt to adjust or operate the Rollator without carefully reading all instructions in this manual.

- **DO NOT** use this product without first reading and understanding the instructions contained in this manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional or dealer before use. Failure to do so may result in bodily injury or damage to the device.
- **DO NOT** self-propel the rollator while seated. Rollators are intended for individual use only and are NOT TO BE USED AS A WHEELCHAIR. Serious injury to the user and/or damage to the rollator may result from improper use.
- **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs. Avoid curbs and other obstacles as serious risk of fall or injury may occur.
- All wheels must be in contact with the floor and/or ground at ALL TIMES.
- **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result.

Maximum weight capacity: 350 lb (158 kg)

Place items completely in the bag/basket so they do not protrude


Excess weight may cause the rollator to tip, resulting in bodily injury or damage to the rollator. The bag/basket has a weight limitation of 10 lb (2,2 kg).

- **DO NOT** perform any adjustments to the rollator while it is in use. Before each use, make sure the rollator frame is fully and securely open.

IMPORTANT SAFETY NOTICE

Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

PART LIST

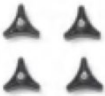 Triangular Knobs (x4)	 Five Point knobs (x2), Screws (x2), Curved Washers (x2)	 Large C Clip (x2) Small C Clip (x2)
 Back rest (x1)	 Rollator frame (x1)	 Bag (x1)
 Handle bars with brake assembly (x2) Back Wheels (x2)	 Front wheels (x2)	

ASSEMBLY

1. Remove all parts from the carton and check that the parts are complete.


Then take out the Main Frame and press down on the seat cushion support tube until the frame is in full extension.

2. Install front wheels.

Prior to assembly, remove four black shipping plugs from frame tubing and discard.


Take front wheel and position the indentation in the leg tubing facing forward, away from rollator seat. Push the leg tubing into the frame tubing.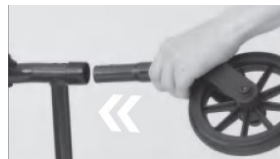

- Line up the top hole on the wheel with the top hole on the frame and push a large C-Clip through. Ensure the C-Clip has been inserted prior to the next step.


- After inserting the large C-Clip, use the triangular knob to screw the front leg in place. Lay the rollator on its other side, then repeat steps for the other front wheel leg. Hand tighten only! Ensure all parts are secured prior to use.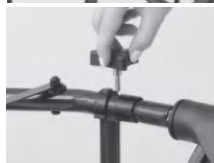

3. Rear Wheels and Brakes


Attach the rear legs to frame. Position the rear wheel and brake shoe on the outside of the rollator. Ensure the indentation in the leg tubing is facing away from the rollator seat and insert into the tubing.

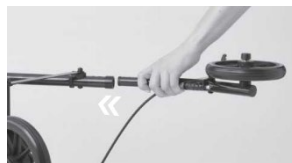- Line up the top hole on the wheel with the top hole on the frame and push the small C-Clip through. Use the triangular knob to secure the rear leg in place. Hand tighten only! Insert the small C-Clip prior to adding the triangular knob.

- On the same side insert the brake assembly into the frame and set to desired height. Brake handle and brake cable must be on the outside of the frame.
- Insert the screw from the inside of the frame into the 6 sided hole. Make sure the screw head fully recesses into the frame.
- Slide the curved washer onto the other end of the screw with the curved side facing the frame.
- Screw on the 5 point knob. Hand tighten only! Ensure parts are secure prior to use.

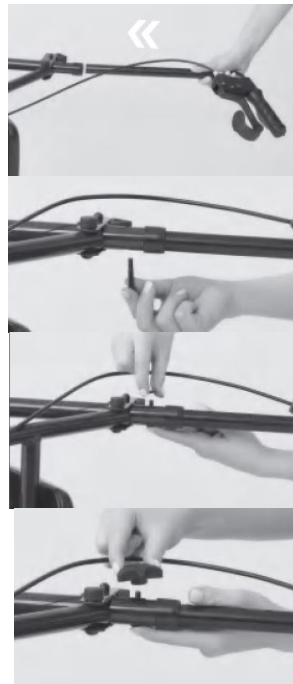- Turn the rollator onto the other side so that the wheels that were installed are now on the floor. Then repeat steps for the other side.

4. **Backrest assembly**

Stand the rollator up so all 4 wheels are on the floor.

- Insert the backrest so that the curved portion faces the front. Make sure that the push buttons snap firmly into the hole. Backrest must be secure prior to use.

- Attach brake cable into clip. If brakes are too tight during assembly, fully extend brake cable to release tension and re-attach to brake clip.

5. Attaching the bag

Loop the cloth hooks onto the seat frame and make sure the bag is centered.

OPERATING INSTRUCTIONS

Handle height adjustment

1. Remove tightening knob from handle.
2. Set handle to desired height.
3. Replace tightening knob into handle, making sure to insert knob from the outside.
4. Repeat Repeat for second handle, making sure both handles are set to the same height setting.

Leg height adjustment

1. Remove tightening knob from leg.
2. Set leg to desired height.
3. Replace tightening knob into leg, making sure insert knob from the outside.
4. Repeat for second, third and fourth legs, making sure all legs set to the same height setting.

Activating the Brakes

1. Locking Brakes
To Slow Down or Come to a Temporary Stop — Pull up slowly on the loop locks with four fingers until the unit slows down or comes to a temporary stop.
2. Parking Brakes
To activate the parking brake, press down on both brake levers until an audible click is observed.
3. To release the parking brake, pull both brake levers back into the original position.

How to adjust the hand brake systems


The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

How to adjust the brake


Step 1: Return both brake levers to their original position.

Step 2: Press the brake pad against the wheel with your hand.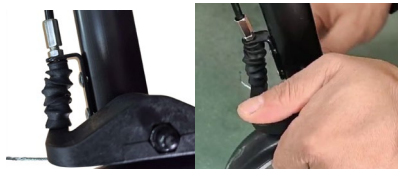

Step 3: Pull the adjustment screw firmly to create a gap between the adjustment nut and the fixed plate.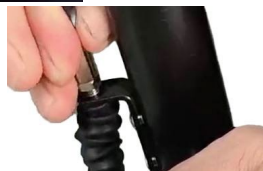

Step 4: Turn the nut clockwise and move it downward to tighten the brake system; turn the nut counterclockwise and move it downward to loosen the brake system.


Step 5: Release the brake pad and test the braking system during motion and while parked.

Hold firmly in place, do not turn.

To tighten the hand brake adjustment. "turn the nut clockwise"

To loosen the hand brake adjustment. "turn the nut counter clockwise"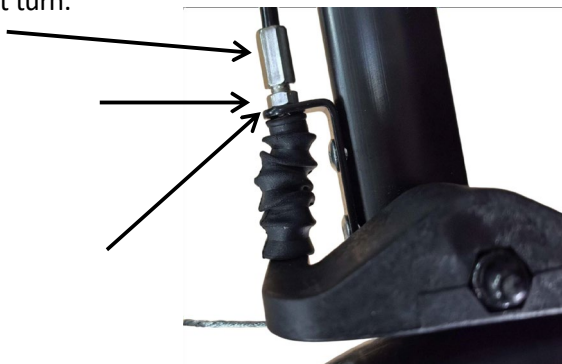

HOW TO FOLD ROLLATOR

To fold, pull up on the folding strap on the seat.

How to Unfold Rollator

To unfold, ensure all 4 wheels are on the ground and press down on the seat cushion support tube. Until an audible click that the frame is in full extension. Keep hands and fingers away from any moving parts!

Warning: Keep your hands and fingers away from any moving parts while unfolding the rollator.

MAINTENANCE

- The frame of the rollator can be cleaned with a moist cloth with mild detergent.
- The wheels can be cleaned with a wet synthetic fiber brush (do not use wire brush).
- Push handles, seat and backrest can be wiped down using mild detergent.
- Do not use aggressive or caustic detergents to clean the rollator.
- Periodically wipe down the rollator with a cloth and non-abrasive cleaner.
- Dry completely before use.

STORAGE & TRANSPORTATION CONDITION

Should be stored in clean, ventilated and dry place, rainproof, moisture-proof, and handle with care during transportation

DISPOSAL

Please dispose the product after use to comply with local regulation.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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