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INTENDED USE

The Rollator is a mobility aid intended to be used by people who are experiencing generalized weakness, difficulty with balance, a lower extremity injury or surgery that need additional support to maintain balance and stability while walking outdoors or used for rehabilitation.

SAFETY INSTRUCTIONS

USER: BEFORE using the Rollator, read this manual and save for future reference.

CAUTION: DO NOT attempt to adjust or operate the Rollator without carefully reading all instructions in this manual.

- **DO NOT** use this product without first reading and understanding the instructions contained in this manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional or dealer before use. Failure to do so may result in bodily injury or damage to the device.
- **DO NOT** self-propel the rollator while seated. Rollators are intended for individual use only and are NOT TO BE USED AS A WHEELCHAIR. Serious injury to the user and/or damage to the rollator may result from improper use.
- **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs. Avoid curbs and other obstacles as serious risk of fall or injury may occur.
- All wheels must be in contact with the floor and/or ground at ALL TIMES.
- **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result.

Maximum weight capacity: 350 lb (158 kg)

Place items completely in the bag/basket so they do not protrude

Excess weight may cause the rollator to tip, resulting in bodily injury or damage to the rollator. The bag/basket has a weight limitation of 11 lb (5kg).

- **DO NOT** perform any adjustments to the rollator while it is in use. Before each use, make sure the rollator frame is fully and securely open.

IMPORTANT SAFETY NOTICE

Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

PART LIST

 Main Frame with bag (x1)	 Handle frame & Rear Wheels (x2)	 Front Wheels (x2)
---	--	--

 Three point short Knobs (x4)	 Three point long knobs (x2)	 Cup Holder (x1)
 Backrest (x1)		

ASSEMBLY

1. Remove all parts from the carton and check that the parts are complete.

Then take out the Main Frame and press down on the seat cushion support tube until the frame is in full extension.

2. Install front wheels.

Insert the front wheel with marking "1" into the frame with marking "1", set to desired height, Secure in place using the three Point Short Knobs, threading the knob in from the outside. Repeat for front wheel "2", making sure both the front and rear legs are set to the same height.

Note: Adjust the position of the leg tube to ensure that the leg tube hole aligns with the frame hole, allowing the knobs to pass through the holes easily.

3. Install rear wheels.

Insert the rear wheel with marking "3" into the frame with marking "3", set to desired height, make sure rear wheels are on the outside, secure in place using the three Point Short Knobs, threading the knob in from the outside. Repeat for rear wheel "4", making sure both the front and rear legs are set to the same height.


Note: Adjust the position of the leg tube to ensure that the leg tube hole aligns with the frame hole, allowing the knobs to pass through the holes easily.

4. Install the handle tube

Insert handle tube into frame and set to desired height. Brake handle and brake cable must be on the outside of the frame. Secure in place using the three point long knobs, threading the knob in from the outside. Repeat for second handle, making sure both are set to the same height.

Note: Adjust the position of the hand tube to ensure that the hand tube hole aligns with the frame hole, allowing the knobs to pass through the holes easily.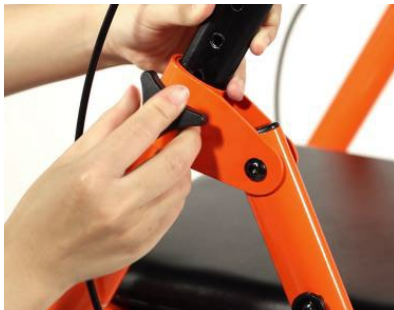

5. Slide & Install backrest

Ensure that the backrest is slid to the very bottom position on the backrest support base. Backrest must be secure prior to use.

6. Fixing brake cable & install the cup holder

Fix the brake cable clamp to the area within 80mm above or below the rear frame cross tube, as shown in the circled area in the diagram below.

Install the cup holder on the plastic mounting seat of the left rear frame tube.

7. Finish installation

Unfold Install the cup holder on the plastic mounting seat of the left rear frame tube.

OPERATING INSTRUCTIONS

Handle height adjustment

1. Remove tightening knob from handle.
2. Set handle to desired height.
3. Replace tightening knob into handle, making sure to insert knob from the outside.
4. Repeat Repeat for second handle, making sure both handles are set to the same height setting.

Leg height adjustment

1. Remove tightening knob from leg.
2. Set leg to desired height.
3. Replace tightening knob into leg, making sure insert knob from the outside.
4. Repeat for second, third and fourth legs, making sure all legs set to the same height setting.

Activating the Brakes

1. Locking Brakes

To Slow Down or Come to a Temporary Stop — Pull up slowly on the loop locks with four fingers until the unit slows down or comes to a temporary stop.

2. Parking Brakes

To activate the parking brake, press down on both brake levers until an audible click is observed.

3. To release the parking brake, pull both brake levers back into the original position.

How to adjust the hand brake systems


The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

How to adjust the brake


Step 1: Return both brake levers to their original position.

Step 2: Press the brake pad against the wheel with your hand.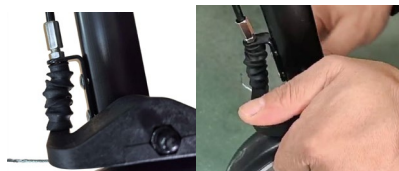

Step 3: Pull the adjustment screw firmly to create a gap between the adjustment nut and the fixed plate.

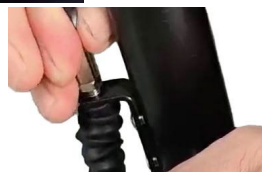Step 4: Turn the nut clockwise and move it downward to tighten the brake system; turn the nut counterclockwise and move it downward to loosen the brake system.


Step 5: Release the brake pad and test the braking system during motion and

while parked.

Hold firmly in place, do not turn.

To tighten the hand brake adjustment. "turn the nut clockwise"

To loosen the hand brake adjustment. "turn the nut counter clockwise"

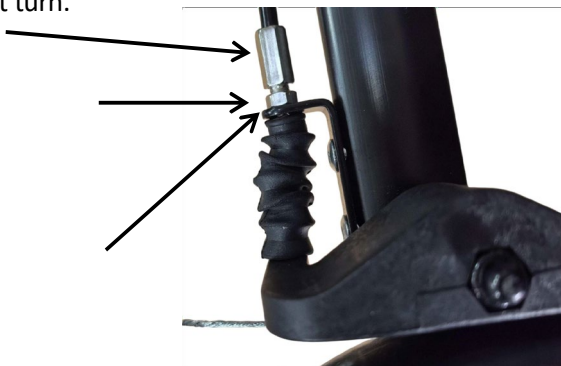HOW TO FOLD ROLLATOR

To fold, pull up on the folding strap on the seat.

How to Unfold Rollator

To unfold, ensure all 4 wheels are on the ground and press down on the seat cushion support tube. Until an audible click that the frame is in full extension. Keep hands and fingers away from any moving parts!

Warning: Keep your hands and fingers away from any moving parts while unfolding the rollator.

MAINTAINENCE

Use a soft dry cloth when wiping the rollator. Keep the rollator clean and tidy.

STORAGE & TRANSPORTATION CONDITION

Should be stored in clean, ventilated and dry place, rainproof, moisture-proof, and handle with care during transportation

DISPOSAL

Please dispose the product after use to comply with local regulation.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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