EN ELECTROMEDICAL LINE

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ES APARATOS ELECTROMEDICALES

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EN Unisonic allows a continuous or pulsated emission which can be set at 25, 50, 75 or 100%. The Unisonic device is electronically controlled by a microprocessor and offers memorized programmes. The device is supplied with multi-frequency transducing handle (1 o 3 MHz frequencies can be chosen). The transducing handle can be used only with this device. It means as a personalized component. It can be used also under water (IP67)

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LTE330
UNISONIC
 Italian, English, and French
 incorporated software
 15 pre-set programmes

TECHNICAL FEATURES

Reference Standards and Directives	
Direttiva	93/42/CEE (D.Lgs. 46/97)
Norma Assic. Qualità	UNI EN ISO 9001(2000) UNI CEI EN 13485 (2002)
Norme di Sicurezza	EN60601-1 EN60601-2-10 EN60601-1-2
Specification	
Feeding	230Vac/50Hz
Power consumption	40VA
Internal fuse	T 3,15mA
Output frequency	from 1 to 3,0MHz
Modulation	25-50-75-100%
Wave form	Continuous or modulated
Modulation frequency	2 impulses/sec
Treatment time	from 1 to 30 min
Output param. indicator	alphanumeric display
Output indicator	visual
Electric safety class	II BF
Protection level	IP20
Protec. level transducer	IP67
Intensity	2W/cm ²
Beam type	collimated

Era (size of transd.)	5 cm ²
Max power (transducer)	10w
Bnr	5

Correlation Table

Indicated Power	Intensity W/cm ²	Total Power W
10%	0,2	1
20%	0,4	2
30%	0,6	3
40%	0,8	4
50%	1,0	5
60%	1,2	6
70%	1,4	7
80%	1,6	8
90%	1,8	9
100%	2,0	10

Wave form	Continuous	Modulated	Modulated	Modulated
Modulation	100%	75%	50%	25%
Repetition Period (μ/sec)	500	500	500	500
Impulse Duration (μ/sec)	500	375	250	125
Efficiency	1,00	0,75	0,50	0,25

Dimension and weight

L: 270 mm

W: 220 mm

H: 110 mm

1,4 kg

EN STANDARD ACCESSORIES INCLUDED
1 feeding cable 2m, 1 handle, 1 handle support

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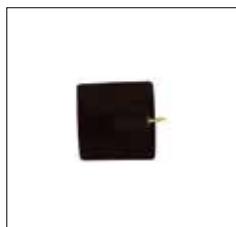
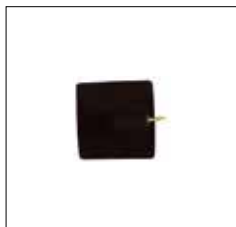
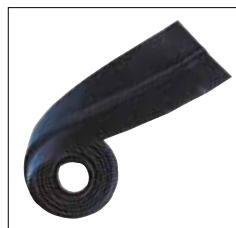
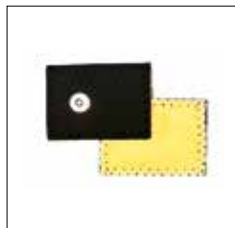
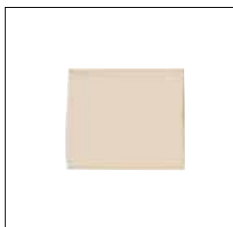
LTR560 3x40cm
LTR562 3x80cm
LTR563 3x100cm
Elastic tissue band

LTR565 8x40cm
LTR567 8x80cm
LTR568 8x100cm
Elastic tissue band

Electrode adapters, cables, and electrotherapy outlet dividers

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
LTR221 Red
LTR220 Black
Cable with 4mm male pin

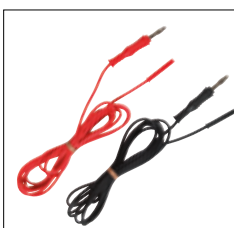LTR222
Red cable for ELESTIM 2
and LEM FITNESS

LTR225R Red
LTR225N Black
Cable for IONOSTIM and
IONODERM

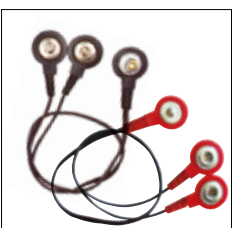
LTR344R Red
LTR344N Black
Male connection clip
adaptor - 4mm

LTR345R Red
LTR345N Black
Female connection clip
adaptor with hole -4mm


LTR346R Red
LTR346N Black
Female connection clip
adaptor with hole - 2mm

LTR347R Red
LTR347N Black
Outlet divider from male
clip to 2 female clips

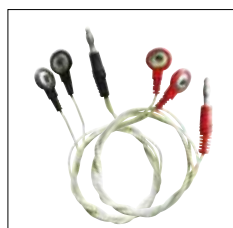LTR348R Red
LTR348N Black
Outlet divider from
2mm female pin to 2
2mm male pins

LTR349R Red
LTR349N Black
Outlet divider from 4mm
male pin to 2 female pins

EN Storage time: these gels don't undergo hydrolysis or oxidation in normal use conditions. A lengthy storage test at about 70°C hasn't caused any alteration of the product

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DRV1800
E.C.G. and TENS gel
 260g dispenser
 Package of 50 pcs
 Carton box of 25 pcs
 Single price

DRV1805
Ultrasound gel
 260g dispenser
 Package of 50 pcs
 Carton box of 100 pcs
 Single price

DRV1806
Ultrasound gel
 1 kg dispenser
 Package of 10 pcs
 Carton box of 15 pcs
 Single price

DRV1807
Ultrasound gel - 5 kg
 260g dispenser to be filled
 Singular packaging
 Carton box of 1 pc
 Single price

DRV1808
Ultrasound gel - 5 kg into soft bag
 260g dispenser to be filled
 Package of 4 bags with 4 dispensers
 and 4 funnels included
 Price for 4 bags