urban

X1T – X7T

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INTRODUCTION

Thank you for choosing an Evoluxion Health product. Our manual wheelchairs are designed to meet a variety of needs and mobility challenges. Please make sure you have read and fully understood the instructions before using the wheelchair. If anything is unclear, reach out to a healthcare professional, authorized dealer, or manufacturer technician for assistance. Failure to do so may result in personal injury and/or property damage.

INTENDED USE

This wheelchair is intended for use at home, in the office, in public areas, buildings, sidewalks, pedestrian paths, and on stable surfaces. It is designed to assist with user mobility and should not be used as a step ladder or to tow or transport objects of any kind.

INDICATIONS

This device is specially designed for individuals who, due to a variety of health conditions, have difficulty standing and/or walking, and therefore require assistance for some or all of their daily activities.

CONTRAINDICATIONS

It is recommended that users be accompanied by a caregiver or assistant. This device is not suitable for individuals with paralysis or other conditions that result in significant loss of body control. These users require specialized wheelchairs that provide additional stability and support.

Caution: Fingers or other body parts of the user or assistant can become trapped between components when folding, unfolding, or adjusting the device. Handle with care during any adjustments. Make sure to check the spacing between device parts to prevent fingers or other body parts from getting caught.

DIMENSIONS

URBAN X1T – X7T		41 cm	43 cm	46 cm	48 cm
Total length (cm)		98	98	98	98
Total height (cm)		89,5	89,5	89,5	89,5
Overall width (cm)		56	58	60	62
Folded width (cm)		41	43	46	48
Seat width (cm)		41,5	41,5	41,5	41,5
Seat Depth (cm)		42-43,5	42-43,5	42-43,5	42-43,5
Backrest Height (cm)		48	48	48	48
Seat Height from Floor (cm)		120	120	120	120
Weight Capacity (Kg)		20	20	20	20
Front Wheel Size ("/cm)		30	30	30	30
Rear Wheel Size ("/cm)		14,8 - 12	14,8 - 12	14,8 - 12	14,8 - 12
Gross Weight (Kg)		79*31*96	79*31*96	79*31*96	79*31*96
Box Dimensions (cm)		98	98	98	98

Frame	1 crossbar, matte green-grey finish
Front Wheel:	8"x2", Polyurethane tire
Rear Wheel:	24" wheel, Polyurethane tire
Armrests:	Flip-back armrests, PU cushion
Footrests	Detachable and swing-away footrests, leg support
Seat & Backrest	Fire-retardant nylon with foam padding
Fork:	Plastic
Footplate	PP
Accessories	Anti-tip

INSTALLATION INSTRUCTIONS

- 1) The initial assembly must be performed by company-authorized technical staff.
- 2) Wheelchairs are typically shipped from the factory folded and packaged, with footrests packed separately. Frame components and spare parts list:

Item	Accessory Name	Quantity
1	Folding frame	1
2	Detachable footrests (with pedals)	1 pair
3	Wrench 1	1
4	Footrest lower strap	1

3) Here are two simple steps to get your wheelchair ready to use:

Step 1: Unfold

Hold both sides of the wheelchair and gently pull them apart so the left and right frames separate slightly. Then, using your hands, press down on both sides of the seat cushion until it is flat. The wheelchair is now open and ready to use. (see Figure 1)

Note: When unfolding, avoid pulling the left and right frames too hard to prevent damaging any parts.

When pressing down the seat cushion, be careful not to get your fingers caught in the seat tube to avoid injury.

Figure 1

Step 2: To install the swing-away footrests, first line up the positioning hole on the footrest with the side bracket on the frame. Then, lift part A and turn it to the right to secure. (See Figures 2 and 3)

Figure 2

Figure 3

Figure 4

Step 3: Adjust the footplate to your preferred height. Hold the safety pin and move the footplate up or down as needed. Once it's set where you want it, tighten the bolt. (See Figure 4)

Using a Wheelchair

1. Manual Operation
 - a. Sit securely in the wheelchair with the brakes engaged and make sure your back, legs, and arms are properly positioned. Only then should the companion unlock the wheelchair and begin moving.
2. Brake Method
 - a. When the wheelchair is stopped, the companion must hold both the left and right brake handles, then push them simultaneously into the locked position. This ensures that the rear wheels are fully locked and prevents them from moving, completing the braking process. finish the braking procedure.
3. If you encounter an obstacle, the attendant should grip both handles firmly, use their foot to lower the rear bottom bar, and gently tip the wheelchair back so the front wheels rise above the obstacle.
4. How to Use the Flip-Up Armrest
 - a. To lift the armrest, press the armrest latch with one hand, raise it from the seat tube, and swing it up. To return it to its original position, lower the armrest and press gently.

1. Folding wheelchair transport method:
First, lift both left and right footrests, then gently raise both ends of the seat cushion with both hands. (see Figure 5)

Figure 5

SAFETY PRECAUTIONS

1) Before driving a wheelchair on your own, practice in a variety of road conditions with healthcare staff present, so you gain enough experience to stay safe.

2) **Requirements for outdoor pavement use**

This product is designed to be used on standard concrete or asphalt surfaces. On unfamiliar terrain or softer ground, be sure to slow down; avoid driving over wet and soft grass, as tires may slip; do not use the chair on loose gravel or sandy beaches; if you are unsure about road conditions, try to avoid them.

3) **Guidelines for Using on Inclines**

When using the wheelchair on a sloped surface, do not tip or make sudden turns. Avoid using the emergency brake abruptly when going downhill, as this could cause the wheelchair to tip forward. Stay away from potentially dangerous slopes, such as those covered in ice, snow, wet leaves, or grass.

4) The brake is meant to lock the wheelchair in place for safety. It should not be used to slow down or stop the wheelchair while it is moving, especially when going downhill.

5) When getting in or out of the wheelchair, raise (fold) the footrest. Do not stand on the footrest when boarding or exiting a bus. Otherwise, the wheelchair may tip forward and cause an accident.

6) Please do not lean forward to pick up items from the ground with your hands.

7) When transferring from the wheelchair to another seat, lean back as much as possible. This will help prevent the wheelchair from tipping over.

8) The wheelchair can safely hold up to 120 kg on flat surfaces. If the user's weight exceeds 120 kg, please use caution or contact the manufacturer.

9) Maintain a consistent speed; it should be kept between 3 and 5 km/h.

10) Operating temperature range: -25 °C to +50 °C; relative humidity: ≤80%; atmospheric pressure: 86 kPa to 106 kPa.

MAINTENANCE - SAFETY CHECKLIST

While using the wheelchair, inspect and maintain it regularly following these steps:

- ☐ Make sure the wheelchair moves straight ahead (don't push or pull it to the side).
- ☐ Check that both the front and rear wheels are in good condition.
- ☐ Be sure the armrest support is securely fastened, reliable, easy to remove, and that all screws are tight.
- ☐ Ensure seat cushions and similar parts are not damaged or misshapen; replace them if needed.
- ☐ Make sure the armrest cushions are comfortable. Wipe down all surfaces of the wheelchair.
- ☐ Check for any rust or corrosion on the metal parts and replace them if they're damaged or corroded.

Note 1: Cushions, handgrips, and armrests can be cleaned with a clean, slightly damp cloth. Do not use organic solvents. If used by patients, manual wheelchairs should be cleaned once a week. If used by patients with infections, clean and disinfect the wheelchair using disinfectants (such as 75% alcohol).

TRANSPORTATION AND STORAGE

This wheelchair is not meant to be used as a replacement for standard transportation. Riding in a wheelchair is not allowed while using any motorized vehicle.

If stairs are needed to move the wheelchair between floors, make sure no one is sitting in it. Use an elevator for anyone riding in the wheelchair to go from one floor to another. If you must use stairs to transport a wheelchair, always remove the occupants and move them separately. Never use escalators to move wheelchairs between floors, as this can lead to serious injury or property damage.

The center of gravity for this wheelchair is located in the middle. Handle with care to avoid injury. Only use secure, non-removable parts to lift or move the wheelchair. Do not use detachable parts for handling—doing so could result in personal injury or damage to the wheelchair.

Storage:

Storage conditions: ambient temperature -40°C to +65°C ; relative humidity: 80%; pressure: 86 kPa to 106 kPa.

Do not store the wheelchair in wet or rainy environments. Exposure to moisture may cause premature rusting or damage to surface finishes, which could result in serious injury and/or property damage.

Do not store or use the wheelchair near open flames or flammable items, as this could result in serious injury and/or property damage.

AFTER-SALES SERVICE

1. After-sales support for this product is provided by Evoluxion Health and its authorized distributors. For warranty-related service, please contact your distributor first. If you are not satisfied with their response, feel free to reach out to us directly. Kindly provide distributor's name, date of purchase, details of the defect, product serial number, and any other relevant information. Replacement parts can be requested or purchased directly from distributors or suppliers.
2. The following situations are not covered by free maintenance:
 - Wear-and-tear parts such as backrest cushions, tires, armrest pads, and plastic components are not included in the warranty coverage.
 - Failures caused by improper use, inadequate maintenance, or incorrect storage not outlined in the manual.
 - Issues resulting from self-assembly or disassembly.
 - Other accidental damage or damage caused by mishandling.
3. Replacement cycle and method for wear parts: It is recommended to replace vulnerable parts such as seat back cushions, tires, hand pads, and plastic components annually.

WARRANTY

This product comes with a three-year warranty starting from the date of purchase. The manufacturer will provide replacement parts for any component that breaks due to manufacturing defects. If you need to make a warranty claim, please contact the distributor where you bought your product. Damage from accidents, misuse, alterations, or normal wear and tear is not covered by this warranty. To activate your warranty, reach out to the distributor where you made your purchase.

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